

REFILLING PT401

The amount of PT401 in the container should be checked weekly. When it is less than half full, you should add more PT401.

1. Remove the cap from the PT401 bottle. It is not necessary to remove the rubber stopper with the hose; this will break the prime of the PT401 pump.
2. Refill the bottle with new PT401 purchased from AmeriWater (P/N 95810125, sold in (4) 1 gallon containers). Close the bottle with the cap.
3. Follow priming procedures above to ensure the PT401 pump remains primed.

PT401 REPLACEMENT PARTS

84-0002	PUMP,WALCHEM, CHEMICAL INJECTOR, EZB SERIES
84521001	VALVE,3-WAY, PRIMING
95810125	PT401, ANTISCALANT (4) 1 GALLON CONTAINERS