

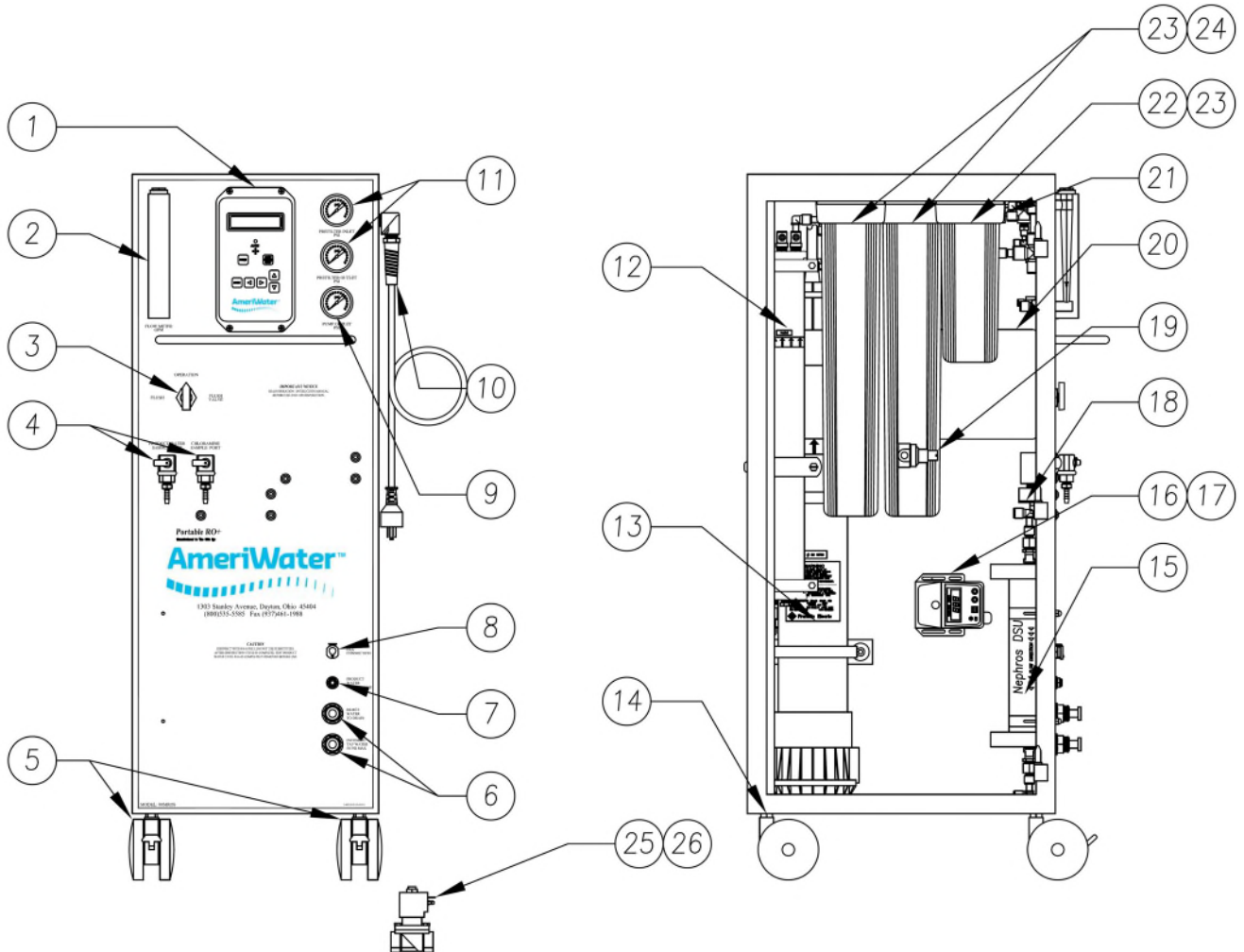



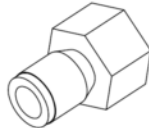

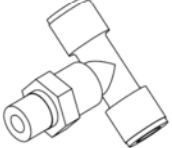

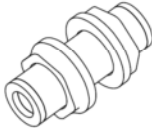

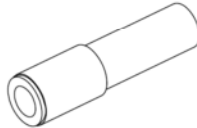

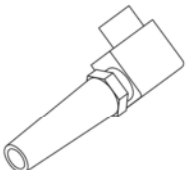
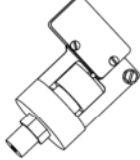
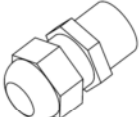

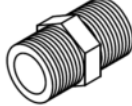


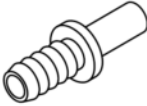


SECTION 10, SPARE PARTS LIST

Item #	Part #	Description	Item #	Part #	Description
1	R69-0019	Controller	15	20-0009	Nephros DSU Filter
2	41530609	Flow Meter	16	R84-0002	Anti-Scalant Pump
3	04-5001	Flush Valve	17	0185-0020	PAA Container
4	041004	Product / Chloramine Port	18	R69-0020	Conductivity Cell
5	94-0013	Caster With Brake	19	R44-0066	Pressure Regulator
6	10-L278	1/2" Bulkhead	20	R61-0008	Motor Starter
7	10-L276	1/4" Bulkhead	21	R65-0014	Pressure Switch
8	16-0042	PAA Connector	22	21-0033	10" Filter Housing (Old Models)
9	430001	Pump Pressure Gauge	23	21530235	O-Ring, Filter
10	66932111	115V Line Cord	24	21-0032	20" Filter Housing
11	430002	Prefilter Gauges	25	R59-0022	Solenoid Valve (NC)
12	R22-2521	Membrane	26	R59-0023	Solenoid Valve (NO)
13	R080-0001	Pump Assembly	N/A	R69-0021	15 PIN Controller patch cord
14	94-0014	Caster	N/A	R69-0022	9 PIN Controller patch cord

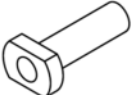

<p>BARB CONNECTORS .25 STEM x .25 BARB P/N: 10521027 .5 STEM X .5 BARB P/N: 10521011</p> 	<p>PLUG 1/4 STEM PN-10-L451 1/2 STEM PN-10521106</p> 	<p>CHECK VALVE TUBE X TUBE 3/8" = P/N 55-0011 1/4" = P/N 55-0015</p> 
<p>CHECK VALVE,.5MPT PN-10720102</p> 	<p>MALE QUICK CONNECTORS P/N: 10-L002 0.25 NPT x 0.25 T P/N: 10-L003 0.25 NPT x 0.38 T P/N: 10-L004 0.38 NPT x 0.38 T P/N: 10-L006 0.5 NPT x 0.5 T</p> 	<p>FEMALE QUICK CONNECTOR P/N: 10-L026 0.25 FPT x 0.25T P/N: 10-L027 0.38 FPT x 0.38T</p> 
<p>SWIVEL EL, NPT x TUBE P/N: 10-L105 0.38 NPT x 0.38 T P/N: 10-L107 0.5 NPT x 0.5 T</p> 	<p>SWIVEL BR TEE, NPT x TUBE P/N: 10-L129 0.25 NPT x 0.38 T</p> 	<p>UNION TEE P/N: 10-L253 0.25 T x 0.38 T P/N: 10-L254 0.5 T x 0.5 T</p> 
<p>QUICK CONNECT BULKHEAD P/N: 10-L276 1/4" P/N: 10-L278 1/2"</p> 	<p>2-WAY DIVIDER P/N: 10-L302 3/8"</p> 	<p>REDUCER P/N: 10-L352 3/8" x 1/4" P/N: 10-L353 1/2" X 3/8"</p> 
<p>PLUG-IN EL, STEM x TUBE 1/4" = P/N: 10-L376 3/8" = P/N: 10-L377 1/2" = P/N: 10-L378</p> 	<p>STRAIN RELIEF ELBOW 1/2" NPT, SPIRAL PN-66510619</p> 	<p>INLET PRESSURE SWITCH P/N: R65-0014 SET AT 3 PSI</p> 
<p>STRAIN RELIEF FITTING PN-66510621 3 HOLE BUSHING PN- 66-0041</p> 	<p>BUSHING .5 MPT x .25 FPT PN-046530201 .5 MPT x .38 FPT PN-046531848</p> 	<p>NIPPLE,.5MPT X CLOSE PN-046531105</p> 
<p>FILTER WRENCH PN-21675184</p> 	<p>HOSES INLET: 0112-0030 PRODUCT: 0112-0070 REJECT: 0112-0065</p> 	<p>GARDEN HOSE ADAPTER, FPT KIT: NUT, WASHER,.5"BARB PN: 0112-0046</p> 

10.1 ROUTINE REPLACEMENT ITEMS (NON-DURABLE COMPONENTS)

PART #	DESCRIPTION
0124-0024	O-Ring Kit, 2.5" Membrane
20-0009	Nephros DSU 510K Filter
20-1011	1 Micron 10" Prefilter
20-1012	1 Micron 20" Prefilter – Serial # 31539 & Above
20-5012	Chlor-plus Carbon Block Cartridge
63-0004	Fuse, MDA-15, 250V, 15 Amp
63-0010	Fuse, MDL-6.25, 250V, 6.25 Amp
95810125	PT401 Antiscalant, (4) 1 Gallon Containers
95810126	PT401 Antiscalant HpH, (4) 1 Gallon Containers
R22-2521	MROS Membrane (2.5" dia X 21")
95-0007	Peracidin Disinfectant, 4 Quarts
000-095-0001	Kit,Biotrol+,5 Gal Container+Powder,Portable MRO
97HP20401	Test Strips Peracid Test (6 Bottles of 100 Strips each) For Measuring High Range Paracetic Acid
97PX20501	Test Strips Renal Check (6 Bottles of 100 Strips each) For Measuring Residual Peracetic Acid
97PH20901	Test Strips pH (6 Bottles of 100 Strips each) For Measuring pH/Water/Acid,Base/Bicarbonate/Dialysate
97RC22101	WaterCheck RC (6 Bottles of 100 Strips each) For Measuring Residual Chlorine
97CM20201	WaterCheck 2 (6 Bottles of 100 Strips each) For Measuring Low Level Chlorine/Chloramine
LAL	LAL Endotoxin Testing, Exact results in just a few days
AAMI	AAMI Chemical Analysis, Results within one week

*Call AmeriWater or your AmeriWater distributor for pricing.